


uTV 3 hurtig guide

Din uTV™ 3 oversigt

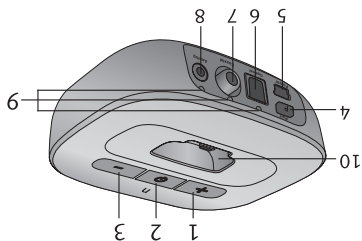

- 1 Volum opp (+) knapp
- 2 Strøm Av/På knapp og hoved indikator lys
- 3 Volum ned (-) knapp
- 4 Parings (P) knapp
- 5 Strømforsyning
- 6 Digital audio (optisk)
- 7 Digital audio (coaxial)
- 8 Analog audio (3,5 mm)
- 9 Audio indikator lys
- 10 Ladebrønn

Forstå indikator lysene

På	Konstant grønt	
På, men ingen audio input	Blinkende grønt	
Paringsmodus	Hurtig blinkende blått	
Streamingmodus	Konstant blått	
Pausemodus	Blinkende blått	
Testmodus	Blinkende flerfarge	

Medus AS
 Postboks 4
 2712 Brandbu
 Telefon 61 32 90 50



Unitron Hearing
 20 Beasley Drive, P.O. Box 9017,
 Kitchener, ON N2G 4X1 Canada



Unitron Hearing GmbH
 Max-Eyth-Straße 20, 70736
 Fellbach-Oeffingen, Germany


Distributed by:
 Hansaton Akustik GmbH
 Sachsenkamp 5
 D-20097 Hamburg, Germany




92189 DD-SP



Steg 1

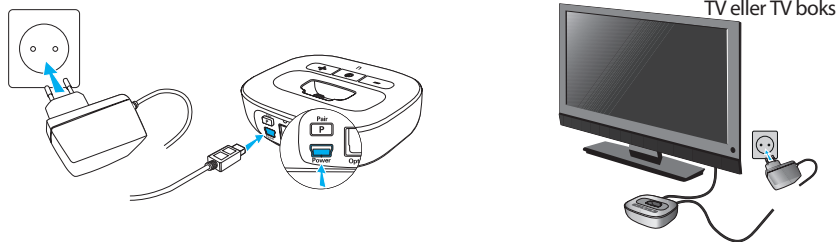

Steg 2

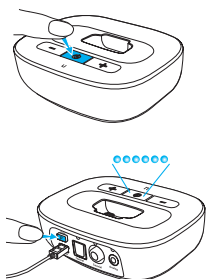

Steg 3


uDirect 2/3




uStream



Steg 4

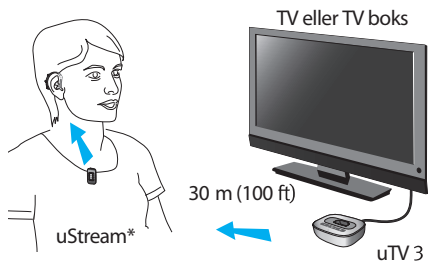

Steg 5

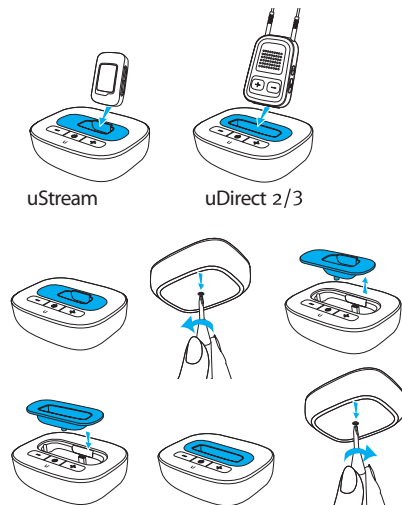

*uDirect 2/3 er også kompatibel

Bruk helst den optiske digitale tilkoblingen hvis mulig.